

# BUSTER BODY SUITS FOR CATS & DOGS

The soft and comfortable **BUSTER Body Suit** is used as protection and comfort after surgery or in case of skin disease. It is an ideal solution for patients recovering from surgery or minor injuries or have tender areas caused by excessive licking, biting or chewing.

## BUSTER Body Suit is perfect for:

- Post operative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skins ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues
- In season



Double fabric under abdomen

## BUSTER Body Suit advantages:

- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Double fabric under the belly enabling pads to be inserted in case of fluid loss, incontinence or when the animal is in season
- Press studs for added security
- Made from 95% cotten, 5% elastane
- Washable at 40°C




## Nature's Call:

- Undo press studs and the velcro tabs and pull back the fabric
- Roll up the fabric under the abdomen and above the back


## Guideline for sizes

Size measured from collar to base of the tail.

Provet Code	Size	Body Length	Suitable for:
BUST BS D1	XXXS	25cm	Small Chihuahua
BUST BS D2	XXS	32cm	Chihuahua, Toy Poodle
BUST BS D3	XS	38cm	Yorkshire Terrier, Toy Poodle
BUST BS D4	S	41cm	Maltese, Pekingese, French Bulldog, Pugs
BUST BS D5	M	45cm	English Cocker Spaniel, Samoyed
BUST BS D6	L	52cm	Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow
BUST BS D7	XL	62cm	Collie, Boxer, Schnauzer, Labrador
BUST BS D8	XXL	73cm	Rottweiler, Golden Retriever
BUST BS D9	XXXL	75cm	Saint Bernard, Newfoundlander

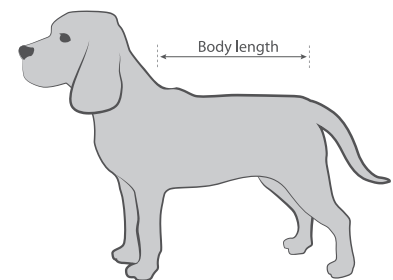

Provet Code	Size	Body Length	Suitable for:
BUST BS C1	XXXS	27.5cm	Small young cat/kitten
BUST BS C2	XXS	33cm	Young cat/kitten
BUST BS C3	XS	38.5cm	Adult cat
BUST BS C4	S	43.5cm	Adult cat, large breed

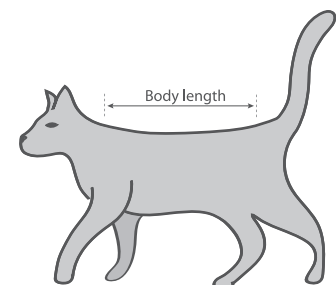

## Instructions for Use:

### Step 1

- Insert the head into the Body Suit
- Insert each front leg separately into the openings provided

### Step 2

- Pull the Body Suit as far as possible over the body

### Step 3

- Put the closing flap between the rear legs and insert the tail through the tail opening
- Attach the closing flap on the velcro tabs on the loin and secure the press studs



A covetrus Company